


Pet CGM

Quick Start Guide



Product composition

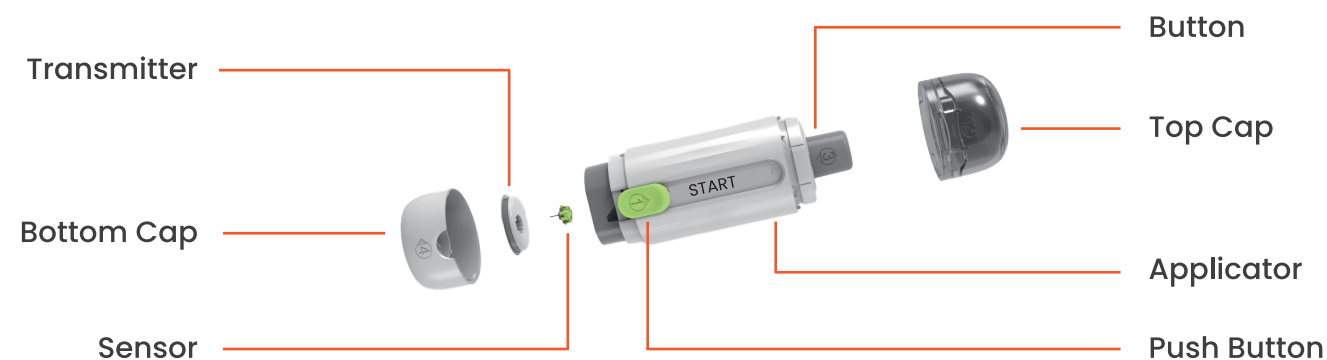

Items you need



Alcohol Wipes/Swabs

Sanitize your pet's skin and wipe-off excessive dirt and oil



Skin-Tac/Skin barrier Wipes

Creates a tacky barrier on your pet's skin and reduce allergy reaction



Pet Trimmer (Recommend over 40 blades)



Tweezer/Hemostates (Helps cgm insertion&removal)



Cohesive Bandage (Helps cgm stay-on)

Service Pack




Overpatch: Helps prevent sensor falling

Steps Overview

- 1 Download APP
Register an account
- 2 Prepping the skin
- 3 Insert Sensor
Connect to the APP
- 4 Monitoring
- 5 Removal

Skin Preparation

If you are using harness for your pets, please be aware to avoid CGM insertion site

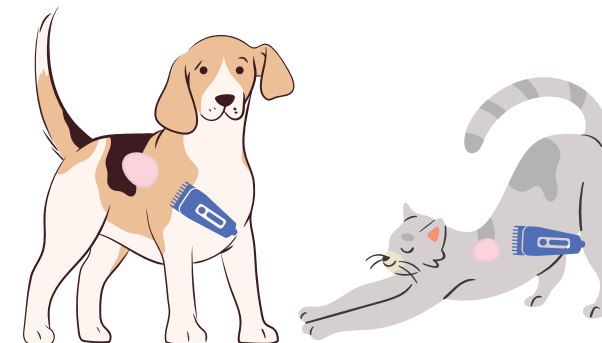

Choose insertion site

Suggest to choose a site generally stays flat with minimal bending or folding of the skin and your pet can hardly scratch or bite.


Shave the area with pet trimmer

Shave a 2-inch to 1.5-inch patch of fur as close to the skin as possible

Clean the site


Use alcohol wipe to sanitize, when the site is dry, wipe the site with skin-barrier Wipe in one direction. Wait until the site is completely dry before inserting the sensor.

Sensor Insertion

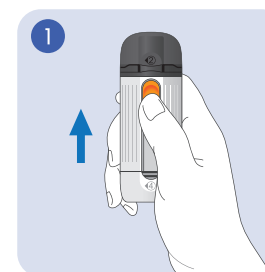

Follow the arrow ①
Push the button upward


Push to the top until you can't push anymore

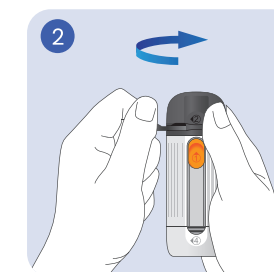

Follow the Arrow ②
Teal off the sealing ring
And open the top cap.




Follow the arrow ③
press down the button
Press until you hear a click sound

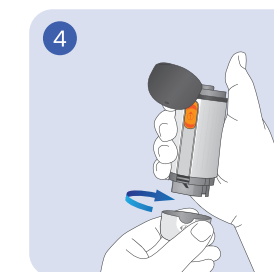


Follow arrow ④
unscrew the bottom cap



Hold applicator against skin.
Press the applicator to the desired area.

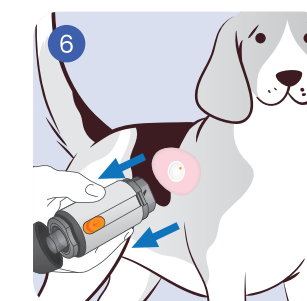

Hold the applicator against pet's skin for 20 seconds.
Then Gently removes the applicator

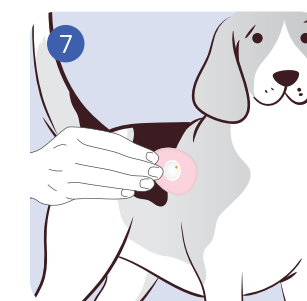

Rub around the adhesive tape to make sure the device is attached firmly

After inserting the sensor, please use FurCarer App to pair your sensor

Secure the Sensor

Recommend Items

Tagaderm / Overpatch

Clothes

Cohesive Bandage

E-Collar/Cone


While our CGM is well tolerated by most pets, the duration of use cannot be guaranteed, because there is a possibility of pets biting or scratching the device off, so it is essential to use protective measures to keep the sensor on.

Each pet has different body shapes, personalities, and activity levels. We can only provide suggestions, and the best reinforcement method needs to be adjusted by the pet owner according to their own pet.

I can bite it off if you don't protect the sensor well



Sensor Removal

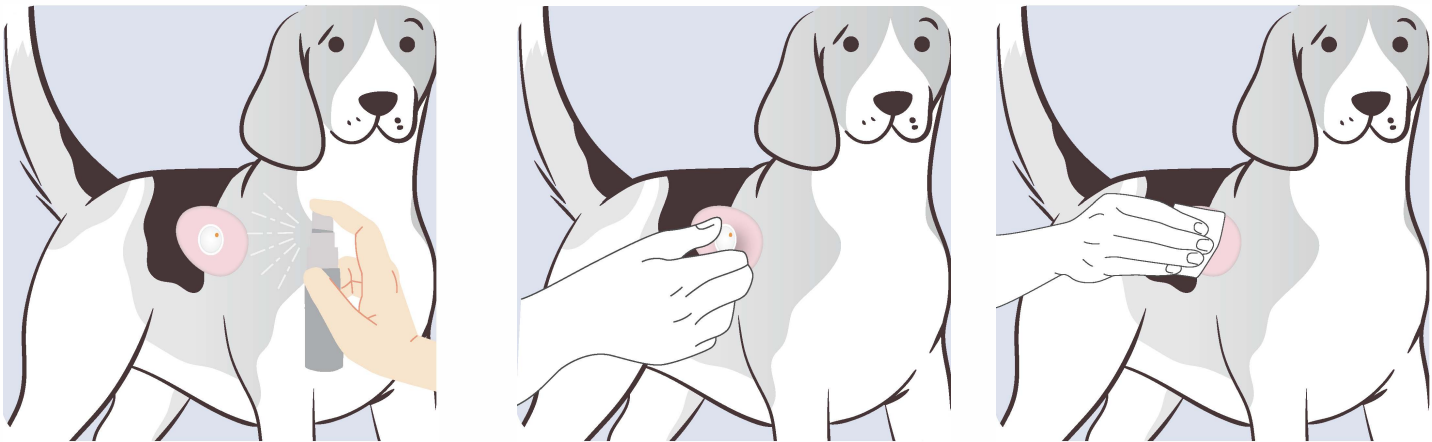

Give a light spray of Furcarer Adhesive Removal Spray (or similar product) to the sensor, and wait for few seconds.

Then use a Q-tip or tweezer to gently removes the sensor off your pet’s skin.

After remove the device, you can give your pet a bath or use wet gauze to completely removes any left residue.

Inappropriate removal of the sensor may cause skin irritation.
Diabetic fur babies’ skin is very delicate!

Warnings

Without confirming glucose readings using a Blood Glucose Meter, Insulin therapy should NOT be changed in the event of unexpected or unusual glucose readings.
Skin irritation may occur from a reaction to the adhesive pad or glue used.
Accuracy may vary in the hypoglycemic range due to limitation of CGM technology.
Protective measures like clothing, harnesses, vests, e-collars, cones, etc. should be used to ensure sensor stays on. While well tolerated by most pets, duration of use cannot be guaranteed and biting or scratching the device off is possible.

Specifications

Sensor Life	Up to 14 days
Reading Intervals	Every 3 minutes
Glucose Range	40-540 mg/dL 2.0mmol/L-30.0 mmol/L
Protection Rating	IP27
Operating Temperature	41°F - 104 °F 5°C- 40°C
Sensor Size	22mm * 22mm
Shelf Life	12 months

Support

Help can be found in the app and in the veterinary website

Website
www.furcarer.com

For further support
E-mail: service@furcarer.com

Download App

Please scan QR Code to download FurCarer App



FurCarer Technology Co., Ltd.
D501-2, 5F, Block D, No. 8 Zhanye Road
Suzhou Industrial Park
Suzhou, 215122, China

FurCarer and any related logos and design marks are either registered trademarks or trademarks of FurCarer
© 2024 FurCarer All rights reserved.